Anwenderhinweise zur Contopharma Comfort-Lösung «drop & see», «drop & see MD»

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Specially designed wetting solution with optimal protection for the eyes.

In clinical trials of practical suitability it was found to be most suitable for all conventional gas-permeable, flexible hard lenses and all conventional hydrophilic soft lenses, especially when replaced regularly (every 1–3 months).

For whom is «drop&see» suitable?

The solution is recommended as a preventive measure for all persons and especially those with dry or slightly irritated eyes that may be stressed by environmental factors such as air conditioning wind, low humidity, air pollution, prolonged working at a computer screen or wearing contact lenses for long periods.

Instructions for use

Drop the solution into the outer angle of the eyelids as needed. It may be used several times daily, even while wearing contact lenses.

Warnings

- The solution must not be used if you are allergic to any of the ingredients. If the conjunctiva becomes red or other eye symptoms arise, you should immediately consult an eye doctor or contact lens specialist.
- Do not use damaged bottles or ophthalmic dispensers.
- After opening the bottle for the first time, do not use the solution for more than 30 days (note the date of first opening on the bottle). Any solution remaining after 30 days can be disposed of as normal. The polyethylene bottle is designed to comply with the requirements of various recycling concepts.
- The ophthalmic dispensers are designed for single use. Preservative free.
- Store the solution at room temperature (15–25°C). Protect from light. To prevent possible microbial contamination of the solution, do not allow the dropper tip of the bottle to touch any surface. Close the bottle immediately after use and always keep it tightly closed.
- If you cannot see as clearly as normal for a short while after using the solution, you should not drive or operate machines until your sight has returned to normal.
- Keep out of the reach of children.

When used as instructed, no side effects have been observed so far with CONTOPHARMA «drop&see» and «drop&see MD» Comfort Solution.

Special instructions for use

Only use the care product and storage container recommended by your contact lens fitter.

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Laboratorium Dr. G. Bichsel AG
Contopharma AG