Manufacturier/Pabrikant / Fabricant
Weissnaustrasse 73 - CH-3800 Unterseen
Vertrieb/Distributeur/Distributore/
Distributor/Distributore/Vertreter
Contopharma AG CH-3800 Interlaken
www.contopharma.ch

Speziell konzipierte Nachbenutzungslösung - Solution
humectante conçue spécifiquement pour le
nettoyage des lentilles de contact et la protection optimale
de l'œil.
Special designed
reworking solution - Soluzione appositamente studiata per
la pulizia delle lenti a contatto e la protezione ottimale
dell'occhio.

CONTOPHARMA
lens&lid
lens & lid MD

www.annecy-lentilles.fr

Deutsch

Anwenderhinweise zur Contopharma Comfort-Lösung «lens & lid», «lens & lid MD»

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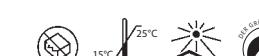
Specially designed rewetting solution with cleaning action on contact lenses and optimal protection for the eyes.

In clinical trials of practical suitability it was found to be most suitable for all common gas-permeable, flexible hard lenses and all common hydrophilic soft lenses.

For whom is «lens & lid» suitable?

The solution is recommended for persons with an increased tendency to develop greasy and smearable contact lens surfaces. It dissolves lipophilic substances and contaminants of the tear film. Ideal for prolonged daily wearing of contact lenses.

Instructions for use

As required, drop the solution into the outer angle of the eyelids. It may be used several times daily, even while still wearing contact lenses.

Warnings

- The solution must not be used if you are allergic to any of the ingredients. If the conjunctiva becomes red or other eye symptoms arise, you should immediately consult an optician or contact lens specialist.
- Do not use damaged bottles or ophthirole dispensers.
- After opening the bottle for the first time, do not use the solution for longer than 30 days (note the date of first opening on the bottle). Any solution remaining after 30 days can be disposed of as normal. The polyethylene bottle is designed to comply with the requirements of various recycling concepts.
- The ophthirole dispensers are designed for single use. Preservative free.
- Store the solution at room temperature (15–25 °C) and protected from the light. To prevent possible microbial contamination of the solution, do not touch the dropper unit on the bottle. Close the bottle immediately after use and always keep it tightly closed.
- If you cannot see as clearly as normal for a short while after using the solution, you should not drive or use machines until your sight has returned to normal.
- Keep out of the reach of children.

When used as instructed, no side effects have been observed so far with CONTOPHARMA «lens & lid» and «lens & lid MD» Comfort Solution.

Special instructions for use

The «lens & lid» solution may also be used temporarily and in exceptional situations as a storage solution for all current types of contact lenses.

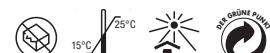
For care and storage of your contact lenses, use care solutions and storage containers recommended by your contact lens fitter.

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Laboratorium Dr. G. Bichsel AG